

PERACIDIN® DISINFECTANT

DESCRIPTION:

Peracidin® is a peracetic acid solution (PAA) for use in reverse osmosis and water system components used in hemodialysis.

Protect the reverse osmosis machine and its distribution system against bacteria, biofilms and endotoxin.

Usually a 1% solution (1 part PAA to 100 parts pure water) is used with a 40 minute soak and recycle time.

Peracidin® is non-toxic and biodegradable, unlike aldehyde vapors. Care to prevent skin bleaching must be taken with the concentrate, but in the 1% dilution form special handling procedures are not required.

Product label includes an expiration date to ensure that the chemical is used at its most effective state. The shelf life of the disinfectant is one year.

Model	Description
95-0006	Quantity of two (2) – Quart Containers
95-0007	Quantity of four (4) – Quart Containers
95-0009	Quantity of twelve (12) – Quart Containers
97HP20401	Peracetic Acid Potency Test Strips (6 bottles of 100 strips each)
97PX20501	Residual Peroxide Test Strips (6 bottles of 100 strips each)

 $\label{eq:peracidin} \textbf{Peracidin}^{\texttt{@}} \textbf{is a registered trademark of Alcavis HDC}.$

